


Quick Guide to Collibio



collibio | 1 | 1.75x | 0°

Gills_II_Haematoxylin-good_detail.tif

Mouse Action

Mouse Action	Description
Click & Drag	Pan
Mouse Wheel Scroll Up	Zoom in
Mouse Wheel Scroll Down	Zoom out
"CTRL" Key + Mouse Wheel Scroll Up	Zoom in (fine grained)
"CTRL" Key + Mouse Wheel Scroll Down	Zoom out (fine grained)
"ALT" Key + Mouse Wheel Scroll Up	Increase z-stack focal level
"ALT" Key + Mouse Wheel Scroll Down	Decrease z-stack focal level
Double Click	Zoom in
"CTRL" Key + Double Click	Zoom out

Key Action

Key Action	Description
Arrow Keys	Pan up, left, down, right
"+"	Zoom in
"-"	Zoom out
"]"	Zoom in (fine grained)
"["	Zoom out (fine grained)
","	Increase z-stack focal level, if applicable
","	Decrease z-stack focal level, if applicable
"O", "Z"	Toggle between navigation mode and annotation mode
"C"	Toggle magniviewer visibility
"V"	Magniviewer visible: Hide magniviewer and focus viewer on magniviewer location. Magniviewer invisible: Show magniviewer at previous location.

Image information

- Image information
- Colour controls
- Snapshot
- Magniviewer
- Toggle thumbnail

Slide thumbnail & viewport location